



# DataRest™ Compliance System

## Quick Guide

Model DRQG

**NOTE:** Check all parts for shipping damage. If damage is found, contact your carrier or homecare dealer.

### Description of Kit

Model Number	Description	Quantity
DRDOC	Docking Station	1
DRC	Data Card	1

### Introduction

The DataRest™ Compliance System makes it easy for your healthcare provider to collect, save, print, and e-mail valuable information collected from the Everest™ 2 CPAP device.


The docking station attaches easily to the bottom of the CPAP device using the same connectors as the humidifier, allowing the data card to retrieve information. The docking station is not intended to be used when the CPAP device is in therapy mode. It is only used for transferring data to and from the data card. The docking station does not have any switches or buttons, and it receives its power from the CPAP device.

Once inserted in the CPAP device, the data card automatically collects and stores CPAP data from the patient or uploads prescription changes or therapy adjustments to the CPAP device. No setup or adjustments are required.

### Device Overview

The DataRest Compliance System includes the following components.

- Docking Station
- Everest 2 CPAP
- Data Card
- Instruction Sheet

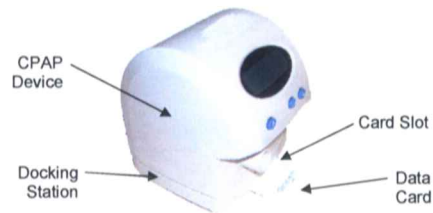

The DataRest Compliance System.

### Connecting the Docking Station

The docking station attaches easily to the bottom of the CPAP device.

1. Unplug the Everest 2 CPAP system from all power sources.
2. Remove the battery and/or humidifier from the CPAP device.
3. Place the docking station on a flat surface; it will tilt upward slightly.
4. Align the recesses on the bottom of the CPAP device with the raised tracks on the docking station.
5. Slide the CPAP toward the rear of the docking station until it clicks into place. The docking station should align evenly with the CPAP device.
6. Reconnect power to the CPAP device.
7. Verify that the docking station is connected properly by waiting until **CARD** is displayed on the CPAP's screen. If **CARD** does not appear on the screen reconnect the power to the CPAP device.

### Inserting the Data Card into the Docking Station

The data card can be inserted in the CPAP device after the docking station is installed.

1. Locate the card slot on the front of the docking station.
2. Insert the data card into the slot completely, making sure to insert the data card face-up in the direction of the arrows.  
**NOTE:** When the data card is fully inserted, information is automatically transferred to the data card.
3. Verify that the transfer is successful by making sure that **Xfer to Card Done** or **RX Done** is displayed on the CPAP screen.  
**NOTE:** When the transfer is finished, the CPAP screen will display **DS** for docking station.
4. Remove the data card from the docking station.
5. Unplug the Everest 2 CPAP system from all power sources.
6. Remove the docking station from the CPAP device.
7. Reconnect the humidifier (if used) and then reconnect the power to the CPAP device.

**NOTE:** If the phrase **NO RX** appears on the CPAP display screen after inserting a data card, contact your home healthcare provider for instructions.

**NOTE:** All of the data card's contents are encrypted, so no patient identification or information is stored on the data card. Only the CPAP device's identification number is stored on the data card, so patient security is never compromised. In addition, prescription changes can only be uploaded to a CPAP device with the matching device identification number.



**AEIOMed, Inc.**  
 1313 5th Street SE  
 Suite 205  
 Minneapolis, MN 55414  
 Phone: 612.455.0550  
 Toll Free: 866.722.2507  
 Fax: 612.455.0551  
 www.aeiomed.com

DataRest™ and Everest™ 2 are trademarks of AEIOMed, Inc. AEIOMed® is a registered trademark of AEIOMed, Inc.