

Questions?

Please call AEIOMed Customer Support at 866.722.2507



AEIOMed®, Inc.
1313 5th Street SE, Suite 205
Minneapolis, MN 55414 USA

Web Site: www.aeiomed.com
Phone: 612.455.0550
Toll Free: 866.722.2507
Fax: 612.455.0551

PART NUMBER: 101144 REV. A

Use and Care Instructions for Your



everest[®]2
HEATED HUMIDIFIER




Grasp the front corners of the humidifier reservoir and slide it from the humidifier base.

To open: Gently lift the acrylic tab at the front of the lid from the reservoir bottom.



To seal: Start from the back of the reservoir bottom. Hook the acrylic lid to the two tabs on the back of the reservoir bottom then bring the lid over the reservoir until it is securely attached. Push the tab at the front of the lid to fully engage the humidifier lid with the bottom.

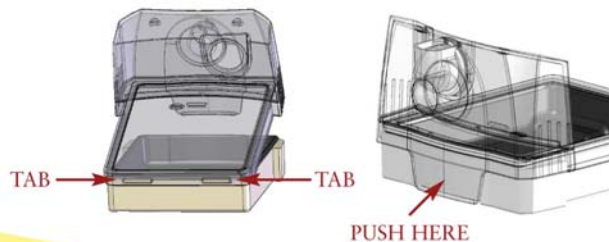

Place the humidifier on end so the hose connector is facing up. Fill the humidifier reservoir to the MAX line with distilled water.



WARNING: Carefully wipe all excess water from the outside of the humidifier reservoir. Pay special attention to the electrical connection on the underside of the reservoir bottom. All surfaces of the humidifier reservoir assembly must be completely dry before using the CPAP.



Place the reservoir into the humidifier base and slide it into position.

Your CPAP and humidifier are now ready for use.

PLEASE REFER TO THE PATIENT MANUAL FOR MORE INFORMATION ON THE HUMIDIFIER ASSEMBLY.